

Directions for use:

- Insert USB into X-Series, power on device.
- From the Supervisor menu, select **Display/Configuration**
- Select **Import Configuration**
- Monitor will display “This will over overwrite...”, select **Yes.**

ZOLL.

X Series® Device Configuration USB Enclosed


KEEP IN A SAFE PLACE

Cloning an X-Series Device

Insert Thumb Drive into X-Series; power on device

- When thumb drive is inserted, a white USB symbol will appear on the screen, just to the left of the battery icon.

Access the X-Series Supervisor Menu

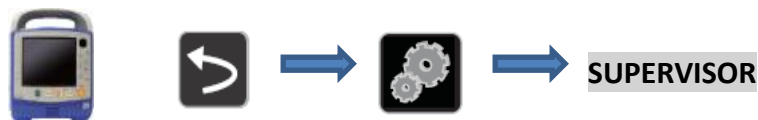

- When prompted, enter your passcode. Your passcode is _____

Download Configuration from USB



- X-Series will display a caution message. Click **YES**

UPDATE DISTRIBUTION LIST FOR 12-LEAD EKG TRANSMISSION (If Applicable)

- Ensure X-Series has an internet connection 

- Select     **Update Distribution List**

- Green light on top of device should illuminate briefly, followed by **Update Successful** on screen.

BE SURE TO UPDATE TIME AND DATE ON X-SERIES